

Key Features

- **DEHP / Plasticiser free**
No plasticiser containing plastic anywhere in fluid / drug path – no risk of plasticiser induced toxicity
- **Latex free**
Reduces the chance of a severe allergic reaction
- **Negligible common fluid space**
Patented multilumen hub keeps infusions separate until distal tip of connector so helping to prevent drug / drug reactions
- **Lipid resistant plastics**
Safe for use with lipid containing drugs
- **Fully rotating male Luer collar**
'Self locates' on locking and prevents line spaghetti

Gravity Line

- **Kink resistant gravity line**
Steel braiding built into catheter wall helps maintain free-flowing drip
- **Low sorbing plastic**
Polyurethane catheter prevents changes in bioavailability of infused drug
- **Low resistance back check valve – opens with minimal pressure head**
Effectively prevents back flow of drugs or blood into gravity line

Drug Line(s)

- **Low sorbing plastic**
Trilayer co-extruded catheter with polyethylene inner lining prevents changes in bioavailability of infused drug
- **Normally closed anti-syphon valve(s)**
One way valve opens with higher crack pressure, effectively preventing accidental overdose from syphoning from pump or syringe
- **Colour coded pinch clamp**
Prevents accidental bolus delivery during syringe handling
- **Detachable long line extensions on drug line(s)**
Permits exchange of pumped infusion for new drug eg at end of operation

A2-Set

Product code – 7200A


Gravity line with back check valve

L – 8 cm, prime vol – 0.5 ml

High flow drug line with colour coded antisyphon valve

L – 10 cm, prime vol – 0.6 ml

Units / box – 50

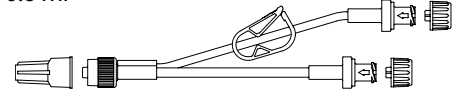

A2L-Set

Product code – 7200L


Gravity line with back check valve

L – 8 cm, prime vol – 0.5 ml

High flow drug line with colour coded antisyphon valve and detachable long line

L – 210 cm, prime vol – 5.1 ml

Units / box – 50

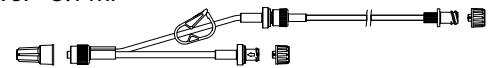

A3-Set

Product code – 7300A

Gravity line with back check valve

L – 8 cm, prime vol – 0.5 ml


High flow drug line with colour coded antisyphon valve

L – 10 cm, prime vol – 0.6 ml

Low volume drug line with colour coded antisyphon valve

L – 12 cm, prime vol – 0.3 ml

Units / box – 50

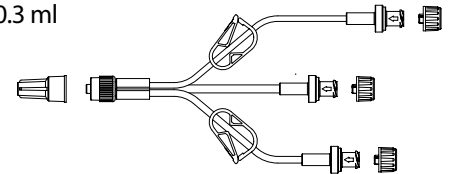

A3L-Set

Product code – 7300AL

Gravity line with back check valve

L – 8 cm, prime vol – 0.5 ml


High flow drug line with colour coded antisyphon valve and detachable long line

L – 185 cm, prime vol – 5.0 ml

Low volume drug line with colour coded antisyphon valve and detachable long line

L – 187 cm, prime vol – 0.68 ml

Units / box – 50

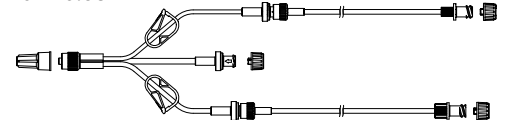

A2NF-Set

Product code – 7200ANF

Gravity line with back check valve

L – 8 cm, prime vol – 0.5 ml

High flow drug line with needle free valve

L – 10 cm, prime vol – 0.6 ml

Units / box – 50

